

Cleaning Information for Dymedix Diagnostics® Reusable Sensor Products

Please find below a list of cleaning / sterilization agents that have been used by customers without damage to the product. Dymedix does not suggest or recommend a specific cleaning agent.

Acceptable Cleaning Agents	Wipe	Spray
Biozide	•	•
CaviWipes®, CaviCide®	•	•
Cidex®	•	•
Control III®	•	•
Hydrogen Peroxide	•	•
Metricide®	•	•
Quat™	•	•
Sani-Cloth® Wipes	•	
Sanizide®	•	•
Soap (mild detergent)	•	•
Sporicidin®	•	•

To clean an item, use a hospital approved non-corrosive cleaning agent according to manufacturer's instructions and your facility's infection control policy.

Air-dry items completely before using on patients



- Do not immerse or soak filtration modules in any cold sterilization liquids.
 - Do not use a steam autoclave method for sterilization.
- Do not pull on wires when wiping airflow or effort interface cables.



Dymedix Diagnostics
5985 Rice Creek Parkway
Shoreview, MN 55126
Phone: 763-789-8280 • 1-888-212-1100
www.dymedix.com