



Product Cleaning Information

Cleaning Instructions for the TriplePlay™ Reusable Airflow Sensors and Perfect Fit 3D™ Effort Modules

These items are considered non-critical devices by the FDA which typically only require a thorough cleaning using a mild detergent and wiping with a cloth to remove any visible filth followed by rinsing with water. Wipe dry or allow to air dry before re-use. If disinfection is required follow your facility's infection control protocols. When using disinfecting wipes, follow the wipe manufacturer's instructions for use. Note that some chemicals may discolor the item. Do not autoclave, doing so will damage the item.

Cleaning for FM™ Airflow Interface Cables with Filtration Modules

Filtration modules are considered non-critical devices by the FDA which typically only require a thorough cleaning using a mild detergent and wiping with a cloth to remove any visible filth followed by wiping with a clean cloth to remove detergent. Do not rinse or submerge in liquid. Wipe dry or allow to air dry before re-use. If disinfection is required follow your facility's infection control protocols. When using disinfecting wipes, follow the wipe manufacturer's instructions for use. Note that some chemicals may discolor the item. Do not autoclave, doing so will damage the device.

Cleaning for Perfect Fit II Reusable Effort Belts

Clean the 8" (20cm) effort sensor and wires by wiping with a cloth to remove any visible filth followed by wiping with a clean cloth to remove detergent. Do not rinse or immerse in liquid. Wipe dry or allow to air dry before re-use. If disinfection is required follow your facility's infection control protocols. When using disinfecting wipes or spray, follow the manufacturer's instructions for use. Do not autoclave, doing so will damage the device.

Clean the reusable belt straps with a non-corrosive hospital approved fabric laundry detergent. Completely rinse the reusable belt strap in water. Machine-washing straps on a gentle cycle is also permitted. Allow the reusable belt straps to dry. Belt straps must be completely dry before using on a patient. Do not use bleach to clean.