EN	<b>Perfect Fit™ 3D Adult Effort Belt System Reusable Effort Module</b>
----	--

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EN
----

#### CAUTION STATEMENTS

This product is for diagnostic purposes only and is not to be used in life supporting or life sustaining situations.

Any serious incident that has occurred in relation to this device should be reported to Dyemedix Diagnostics and the competent authority of the Member States in which the user and/or patient is established.

Symbol	Definition of Symbol
	Latex Free
	Do not re-use
	Federal Law (USA) restricts this device to sale by or on the order of a physician.
	Medical Device
	Consult instructions for use.
	Date of Manufacture Expressed as: YYYYMMDD
	Manufactured by

#### FEATURES AND INTENDED USE

The Dyemedix Perfect Fit™ 3D Reusable Effort Module has been designed to connect with Dyemedix disposable button-hole effort straps and to the PSG head box during sleep disorder studies for recording onto a physiological recorder. Features and specifications of this device are subject to change without notice. This Perfect Fit™ 3D Reusable Effort Module must be used with the Dyemedix disposable button-hole effort straps.

#### INSTRUMENT CONNECTIONS & SETTINGS

Refer to the polysomnograph operator’s manual for direction in determining proper instrument connection and settings. Response is affected by variables such as sensitivity settings, high and low frequency filter settings, sensor placement, patient movement, etc.

#### PRECAUTIONS

- Store Reusable Effort Module at room temperature.
- Do not autoclave the Reusable Effort Module.
- If product is damaged, discontinue use.
- Use all available recorded parameters in making diagnosis.
- Do not apply electrode to broken or damaged skin.

#### REMOVAL AND CLEANING INSTRUCTIONS

- Disconnect Reusable Effort Module leads from recording equipment.
- The Reusable Effort Module may be wiped clean with a hospital-approved disinfectant.
- The Reusable Effort Module may be immersed and soaked in a hospital-approved cold sterilization / disinfectant solution. Make sure the Reusable Effort Module is completely dry before using.
- Disposable button-hole straps used with the Reusable Effort Module are single use and can be discarded.
- If there is no signal output from the sensor and it is properly placed on the patient discontinue use.

#### PATIENT APPLICATION

- Plug the blue/black Reusable Effort Module connectors into the chest/thoracic effort inputs on the head box. Plug the yellow/black into the abdomen/abdominal inputs.
- Incorrect connection could result in erratic or no signal.
- Obtain and cut a section of Dyemedix disposable button-hole effort strap material to an appropriate length. Wrap the strap around the patient’s chest or abdomen and connect the strap to the effort module using the buttons.
- Proceed with polysomnography as outlined by clinician procedure.

#### WARRANTY

Dyemedix warrants its sensors to be free from defects in materials and workmanship and to perform satisfactorily for One (1) Year from the date of purchase. Should the Reusable Effort Module fail during the warranty period, the sole liability of Dyemedix is limited to replacement of the product with no charge for parts or labor. Under no circumstances shall Dyemedix be liable for any consequential damages arising from the use of, or the inability to use this product. This warranty is rendered void for conditions resulting from: damage, marginal performance or malfunctions caused by: misuse, abuse, neglect, improper amplification settings, improper line voltage, power fluctuations, or any adverse environmental conditions, tampering, unauthorized modifications, adjustments or repairs to the product or its accessories. This warranty is in lieu of all other warranties, expressed or implied and is extended only to the original purchaser.

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MedEnvoy Global B.V. Prinses Margrietplantsoen 33 Suite 123 2595AM The Hague The Netherlands